


Instructions for Eaglesoft Integration

Step 1 – Download the DentCapture Software

- Use this link to begin the download: [DentCapture](#)
- Once you install the software, right click on the DentCapture icon in the lower right corner of your screen or click on the caret symbol to expand the system tray.
- A menu will display. Click others and select **Eaglesoft**.

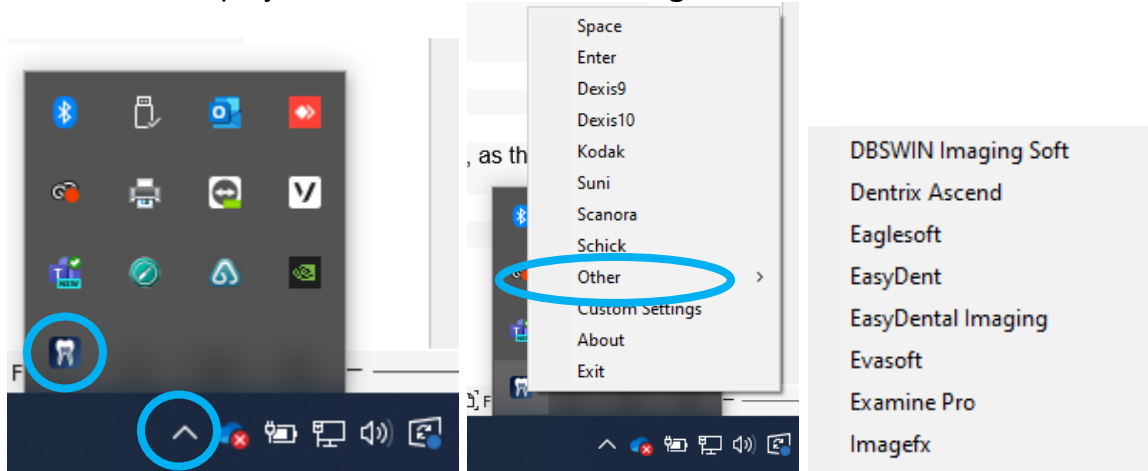

Step 2 – Plug in the intraoral Camera


Make sure that you plug the USB connector into a powered USB port. The camera light should flash when the device is plugged in. If there is no light, try using a different USB port.

Step 3 – Windows Recognizes the Camera

Once the Intraoral camera is plugged in, you should see a small alert on the bottom saying that New Hardware **USB Video Device** was recognized.

Step 4 – Set Eaglesoft Preferences


From the Eaglesoft home screen, click **FILE > PREFERENCES** and you will see this screen:

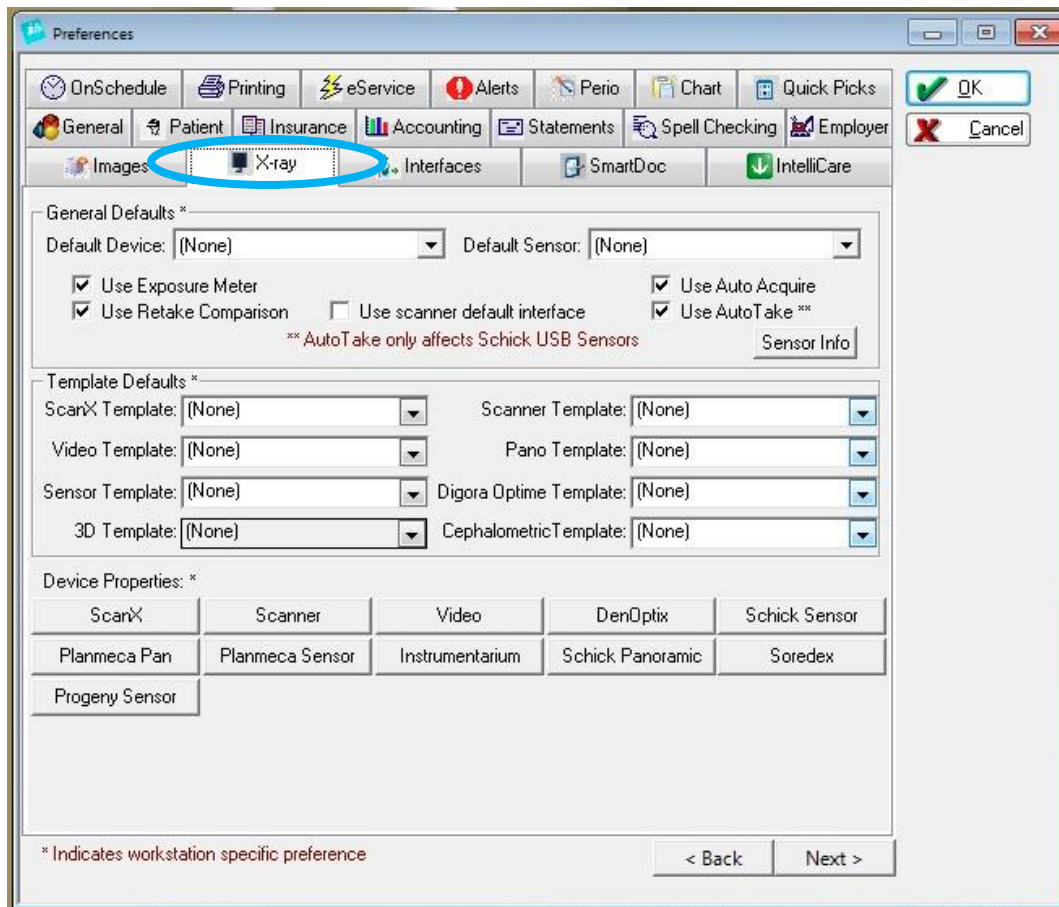


- Click on the **X-Ray** tab

NOTE: Do not change the Default Device and Default Sensor settings, as these are linked to your digital x-ray sensors.

- Click on the **Video** button:

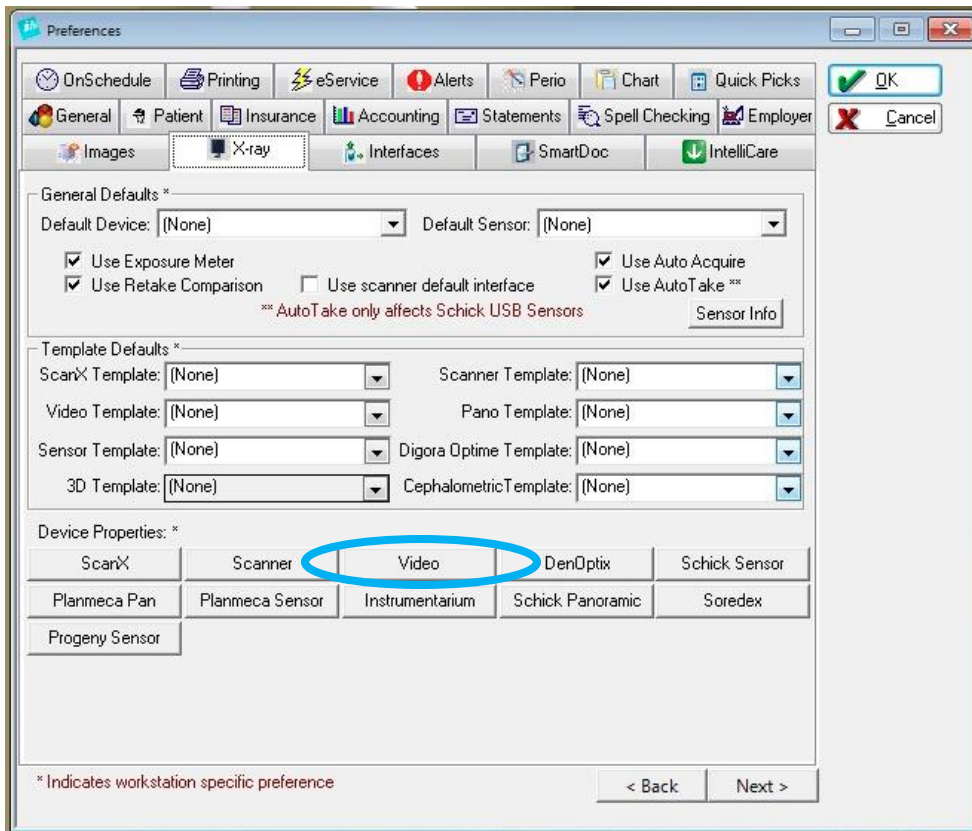


- Select USB Video Device to activate the intraoral.

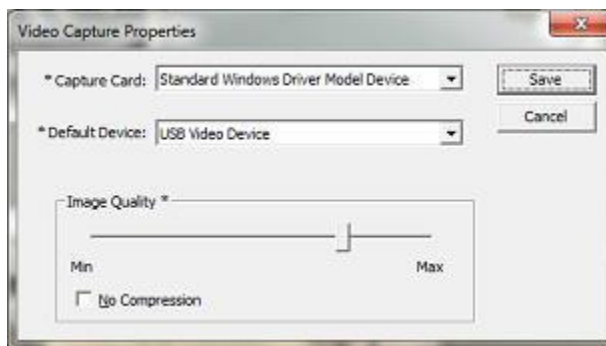

Select **Standard Windows Driver Model Device** as the Capture Card and **USB Video Device** as the Default Device. Click **Save**.

- This will take you back to the preferences window. Click **OK**

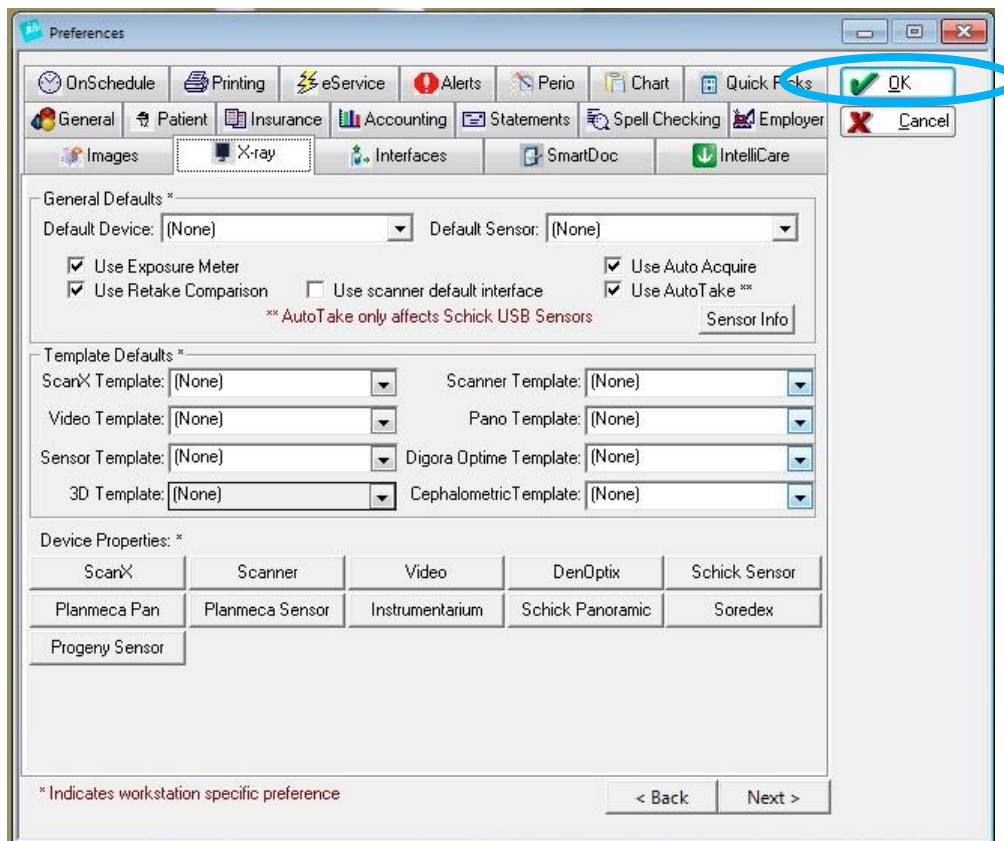

Step 4 – Log off Eaglesoft and Log In

In order for the changes to set, you will need to log off and log back into Eaglesoft.

Step 5 – Start using your intraoral camera!

Click the camera wand icon on the “Operator” screen. This will take you right into an exam where you can capture as many still photos as you like. When finished, click Cancel. The images taken should go right into the patient’s record.

Having issues? Call our support to schedule an appointment! 1-877-5093180.