


Instructions for Dexis 9 Integration

Step 1 – Download the DentCapture Software

- Use this link to begin the download: [DentCapture](#)
- Once you install the software, right click on the DentCapture icon in the lower right corner of your screen or click on the caret symbol to expand the system tray.
- A menu will display. Click **Dexis 9**.

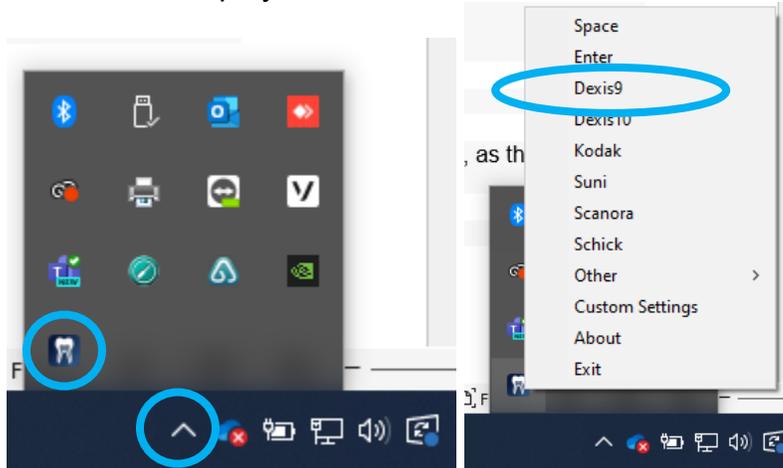

Step 2 – Plug in the Intraoral Camera


Make sure that you plug the USB connector into a powered USB port. The camera light should flash when the device is plugged in. If there is no light, try using a different USB port.

Step 3 – Windows Recognizes the Camera


Once the ICam camera is plugged in, you should see a small alert on the bottom saying that New Hardware **USB Video Device** was recognized.

Step 4 – Set Dexis Preferences

From the Dexis home screen, click the **wrench** icon at the top right.

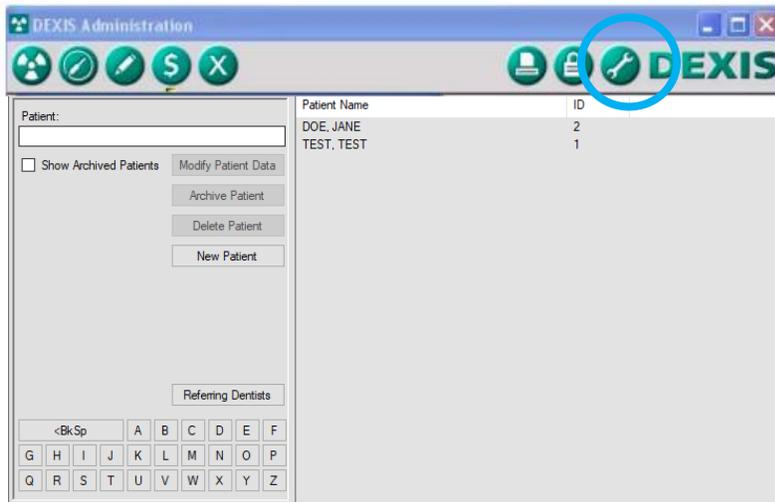

- A Tools and Preferences window will open
- Select **DEXImage** tab on the screen
- Under Video Hardware, select **WDM 640 x480**
- Under Capture Hardware, **USB Video Device**
- Click **DONE**.

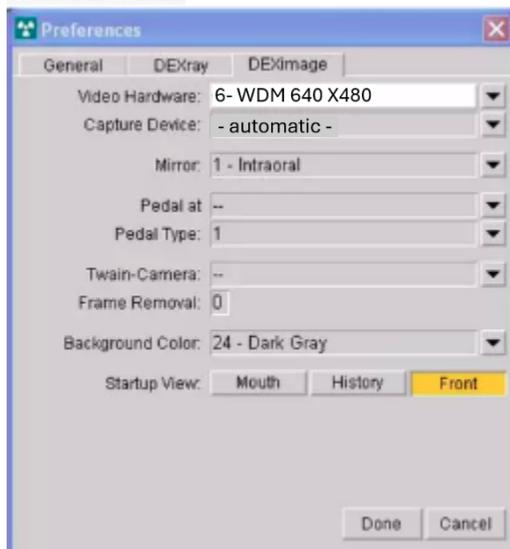

Step 5 – Start using your intraoral camera!

Inside the patient record, click the intraoral  camera icon , and the live video window will appear. You may now start using the intraoral camera.

***Dexis ver. 9.4.5** crashes Dexis for all cameras when launched. Copy and paste VIDEO32.DLL into C:\DEXIS. If it still does not work, contact Dexis support.

Having issues? Call our support to schedule an appointment! 1-877-5093180.