


Instructions for Dentrrix Ascend Integration

Step 1 – Download the DentCapture Software

- Use this link to begin the download: [DentCapture](#)
- Once you install the software, right click on the DentCapture icon in the lower right corner of your screen or click on the caret symbol to expand the system tray.
- A menu will display. Click **Dentrrix Ascend**.

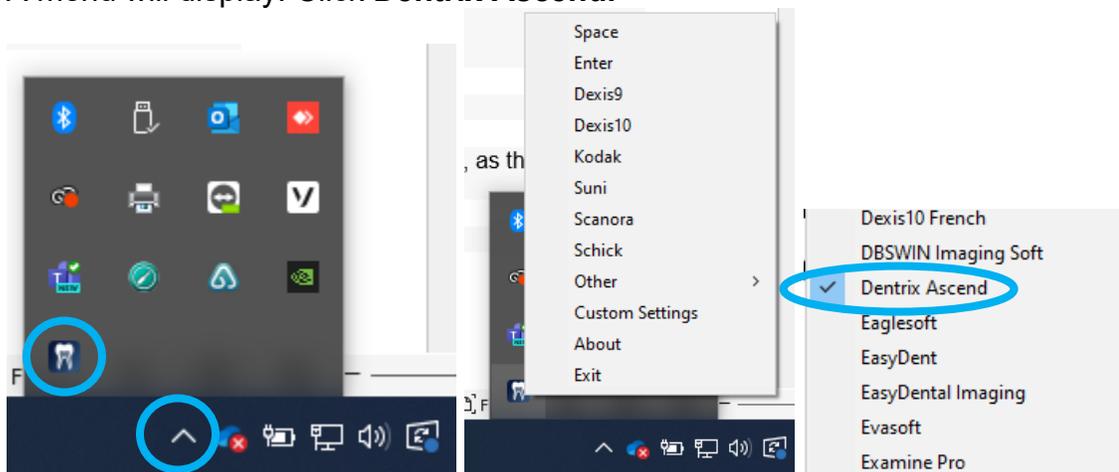

Step 2 – Plug in the intraoral Camera

Make sure that you plug the USB connector into a powered USB port. The camera light should flash when the device is plugged in. If there is no light, try using a different USB port.


Step 3 – Windows Recognizes the Camera

Once the Intraoral camera is plugged in, you should see a small alert on the bottom saying that New Hardware **USB Video Device** was recognized.


Step 4 – Configure Dentrrix Ascend

Select a patient. Click **Imaging > Acquire Images > Standard**.

Select **2D Oral/Facial Photos**. Select **Generic Intraoral Camera**. Click **Proceed with Acquisition** at the bottom.

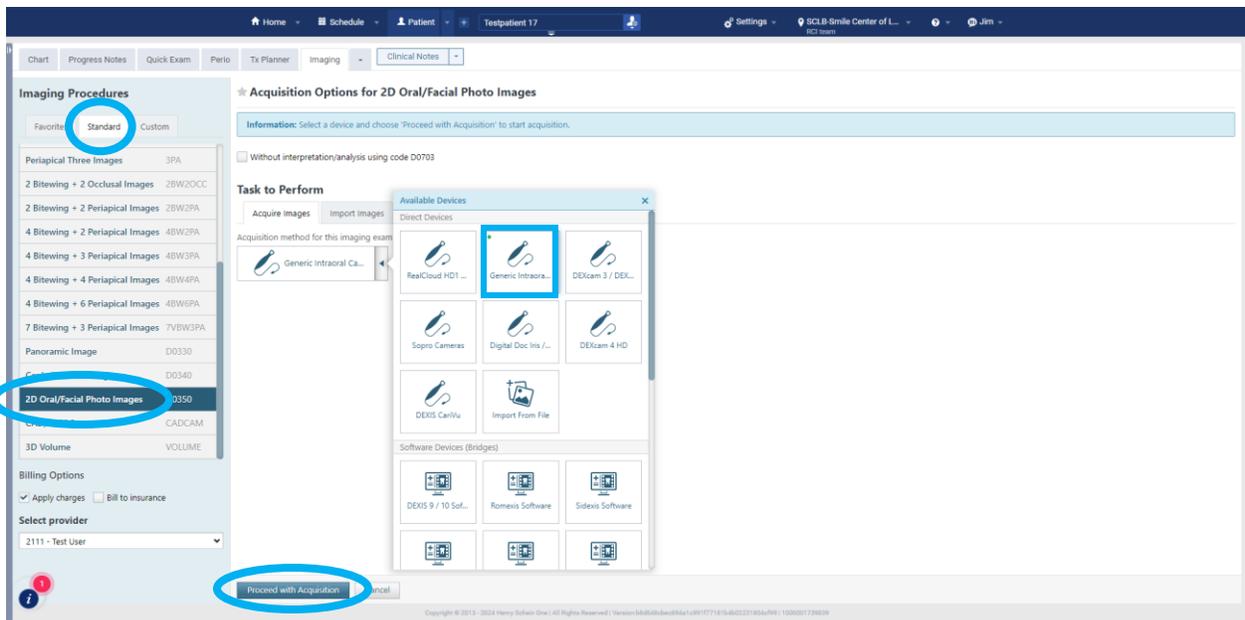

If camera access is blocked, click the view site information icon beside the website address. Click Site Settings.

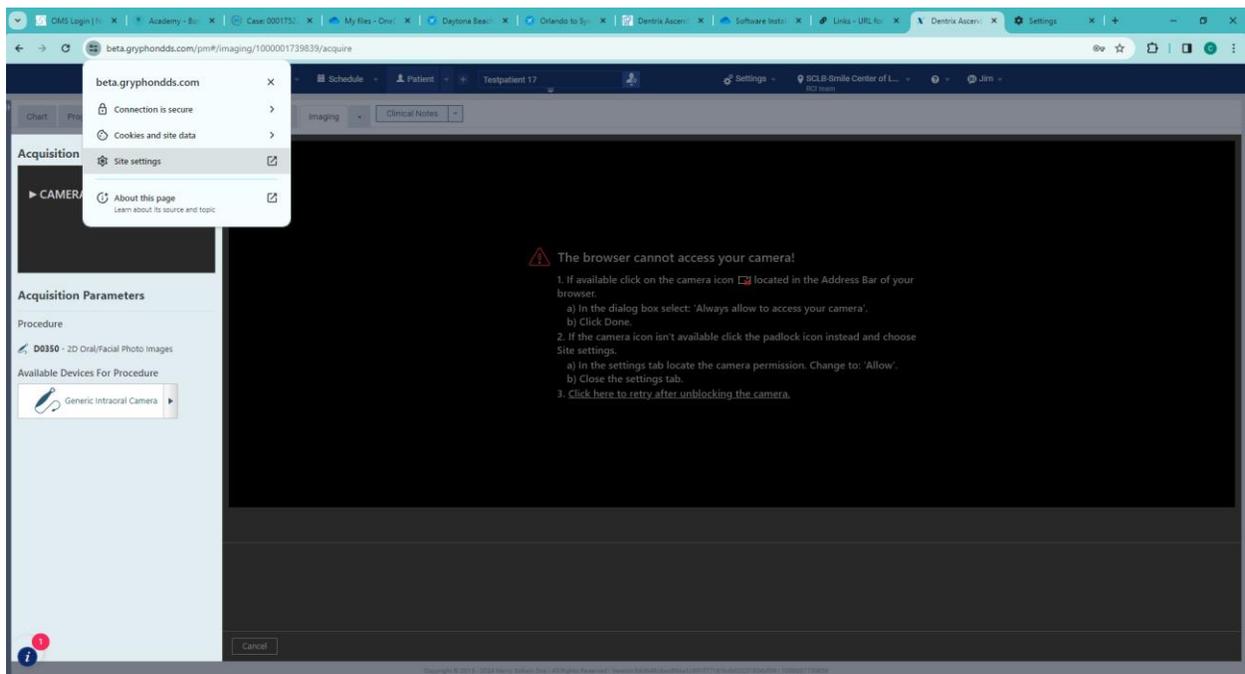

In the camera permission, select **ALLOW**.

Usage

3.9 MB · 15 cookies

Delete data

Permissions

Reset permissions

 Location

Ask (default) ▼

 Camera

Allow ▼

Reload/ refresh webpage to accept changes.

Step 5 – Start using your intraoral camera!

Having issues? Call our support to schedule an appointment! 1-877-5093180.