

Intraoral Camera installation in Vixwin

1. Download dentcapture.exe from <https://denterpriseintl.com/download-drivers/>
2. Install the dentcapture software (run as admin)
3. On the system tray, right click the dentcapture icon and select "other" then "VixWin".
4. Plug in the intraoral camera.
5. Go to Vixwin Imaging and open a test patient.
6. Open the Camera Control Window. Select USB Video Device.
7. Start capturing by clicking the button.
8. All set!

*Please make sure that Dentcapture icon is running in the system tray and the imaging software is correctly selected

*VixWin tech notified us that 3rd party cameras won't work with VixWin Platinum running on Windows 10, but this info needs verification